



GREEN&CLEAN RD N

Dosing table

Liter	0.25%	0.5%	1.5%	2.0%	4.0%
1	2.5 ml	5.0 ml	15.0 ml	20.0 ml	40.0 ml
2	5.0 ml	10.0 ml	30.0 ml	40.0 ml	80.0 ml
3	7.5 ml	15.0 ml	45.0 ml	60.0 ml	120.0 ml
4	10.0 ml	20.0 ml	60.0 ml	80.0 ml	160.0 ml
5	12.5 ml	25.0 ml	75.0 ml	100.0 ml	200.0 ml
6	15.0 ml	30.0 ml	90.0 ml	120.0 ml	240.0 ml
7	17.5 ml	35.0 ml	105.0 ml	140.0 ml	280.0 ml
8	20.0 ml	40.0 ml	120.0 ml	160.0 ml	320.0 ml
9	22.5 ml	45.0 ml	135.0 ml	180.0 ml	360.0 ml
10	25.0 ml	50.0 ml	150.0 ml	200.0 ml	400.0 ml

For the preparation of a ready-to-use surface disinfection solution. Dilute concentrate with water to the preferred concentration (eg. 0.5% dilution + 5 l water = 25 ml concentrate + 5 l water). Use cold water only. After use, discard the used disinfection solution.

Spectrum of efficacy	Dilution ratio	Action time
limited virucidal (e.g. HBV/HCV/HIV, HSV-I)	0.5%	30 Minuten
	1.0%	15 Minuten
tuberculocide	2%	1 Stunde
	3%	30 Minuten
	4%	15 Minuten

Spectrum of efficacy	Dilution ratio	Action time
bactericide, fungicide (C. albicans)	0.25%	4 Stunden
	0.5%	1 Stunde
	1.0%	30 Minuten
	1.5%	15 Minuten