
GREEN & CLEAN AD

METASYS

DE Achtung
EN Warning
FR Attention
IT Attenzione
SV Varning
SL Pozor
FI Varoitus
NO Advarsel

DA Advarsel
HU Figyelem
ET Hoiatus
LV Uzmanību
LT Atsargiai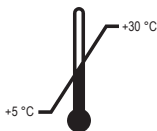

UFI: FV00-E0WD-300Q-FCRV

e 5000 ml
CE 0044

RD-54.008/04
2019-07

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EN: Highly effective, ready-to-use solution for the rapid and sustained immersion disinfection of all prosthetics, dentures as well as dental impressions made of silicone, polyether and alginate. Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, HSV-1); 5 min.; bactericidal, fungicidal (C. albicans); 5 min.; tuberculocidal; 5 min. Application: GREEN&CLEAN AD is ready-to-use. Rinse impression with water directly after removing it from the patient's mouth and insert into the solution. After elapsed contact time (5 min.), rinse again with water to remove remaining disinfectant. The solution has an excellent cleaning power, therefore no additional cleaning agents are required. Standing time: 1 week. In case of strong organic load it is recommended to renew the solution more frequently. Close container during action time and standing time, to prevent evaporation. **WARNING:** Causes skin irritation. Causes serious eye irritation. Harmful to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Composition: 100 g contain: alkylamine (1-2g), dialkylmethyloxyethylammoniumpropionate (0-1g), alkylpolyglycoside (0,2g), ethanolamine (0,3g), water

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**METASYS Medizintechnik GmbH**