GREEN & CLEAN CL

Cleanser for evacuation systems

DE Gefahr	CS Nebezpečí
EN Danger	SK Nebezpečenstvo
FR Danger	SI Nevarno
IT Pericolo	HR Opasnost
ES Peligro	HU Veszély
PT Perigo	PL Niebezpieczeństwo
NL Gevaarlijk	ET Ettevaatust
SV Fara	LV Bīstami
FI Vaara	LT Pavojinga
NO Faretype	RU Опасно
DA Farlig	RO Pericol
EL Κίνδυνος	AR خطر

Produktidentifikatoren / product identifiers:

Alkyl polyethylene glycol ether [69227-22-1],
Subtilisin [9014-01-1]

CE

UN 3267

+30 °C

RD-54.098/01
2018-11

e 500 ml

+0 °C

LOT

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EN Actively defoaming concentrate for cleansing and deodorising evacuation systems.

Daily use with the mixing container: It is recommended to clean the suction system and the spittoon 1-2x a day. 1. Dosage: Open the bottle, fill the dosing cap up to one-fourth (= 5 ml) with GREEN&CLEAN CL and pour into the mixing container. 2. Mixing: Fill the mixing container with water up to the indicator. 3. Suction: Hold small suction tube on opening no. 1 until air is sucked in. Hold large suction tube on opening no. 2 until air is sucked in. 4. Cleaning the spittoon: Pour the remaining solution into the spittoon bowl. Do not rinse.

DANGER: Harmful if swallowed. Causes severe skin burns and eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protec-

tion. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF INHALED: Remove person to fresh air and keep at rest in a position comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If experiencing respiratory symptoms: Call a POISON CENTER or doctor. Store locked up.
Composition: Subtilisin, alkyl polyethylene glycol ether

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