
GREEN & CLEAN RDN

METASYS

DE Gefahr
EN Danger
FR Danger
IT Pericolo
ES Peligro
PT Perigo
NL Gevaarlijk
SV Fara
FI Vaara
NO Faretype
DA Farlig

EL Κίνδυνος
PL Niebezpieczeństwo
CS Nebezpečí
SK Nebezpečnosť
SI Nevarno
HR Opasnot
HU Veszély
ET Ettevaatus
LV Bīstami
LT Pavojinga

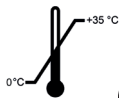UN 1903

e 5000 ml

CE 0044

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EN GREEN&CLEAN RD N

Material-friendly concentrate for disinfection and cleaning of all washable surfaces, floors and medical devices. Aldehyde- and phenol-free. Also suitable for plexiglass surfaces. Alcohol-free.

Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, HSV-1): 0.5% dilution: 30 min., 1%: 15 min.; bactericide, fungicide (C. albicans): 0.25%: 4 hours, 0.5%: 1 hour, 1%: 30 min., 1.5%: 15 min.; tuberculocide: 2%: 1 hour, 3%: 30 min., 4%: 15 min.

Application: Use the METASYS measuring beaker to mix a 0.25% to 4% dilution of GREEN&CLEAN RD N. Leave surface to dry on its own without wiping it again.

DANGER: Harmful if swallowed. Causes severe skin burns and eye damage. May cause damage to organs through prolonged or repeated exposure. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get medical advice/attention if you feel unwell. Store locked up.

Composition: 6% didecyldimethylammonium chloride, 5.5% lauryl dipropylene triamine, 4.2% guanidinium acetate

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