GREEN&CLEAN WD

METASYS

Schnellwirkende Desinfektionstücher
Fast acting disinfecting wipes
Salviette disinfettanti
Lingettes de désinfection

mit Orangenduft
 orange scented
 con profumo d'arancia
 au parfum d'orange

- DE** Achtung
- EN** Warning
- FR** Attention
- IT** Attenzione
- ES** Atención
- PT** Atenção
- NL** Waarschuwing
- SV** Varning
- FI** Varoitus
- NO** Advarsel
- DA** Advarsel
- EL** Προσοχή
- PL** Uwaga
- CS** Varování
- SK** Pozor
- SL** Pozor
- HR** Pozor
- HU** Figyelem
- ET** Hoiatus
- LV** Uzmanību
- LT** Atsargiai
- IS** Varúð
- RO** Atenție
- AR** تنبيه

120

Desinfektionstücher
 Disinfecting wipes
 Salviette disinfettanti
 Lingettes de désinfection

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FR Alcoholic disinfecting wipes from tearproof fleece for disinfecting small surfaces of medical devices (e.g. outer surfaces of straight and contra angle handpieces). Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, Herpes-simplex-Virus Type I): 30 seconds; bactericide, fungicide (C. albicans): 1 minute; tuberculocide: 5 minutes. Application: Completely wet the surface to be disinfected, and keep moistened during the entire action time. Directions for refill: Open the box by a short counter-clockwise turn of the lid. Open the refill pouch and form a roll with its content. Put the roll into the box. Pull out the first wipe from the inner end of the roll and pull it through the opening of the lid. Now the box can be closed again with the lid. Please keep the box closed after use. WARNING: Flammable liquid and vapour. Causes serious eye irritation. Harmful to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Composition: One wipe contains 730 mg ethanol, 245 mg isopropyl alcohol, 3 mg quaternary ammonium salts, 3 mg glyoxal, 645 mg exp.

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