


GREEN&CLEAN BR

Application using a water bottle system



1. Pour GREEN&CLEAN BR into empty dental unit water bottle.*

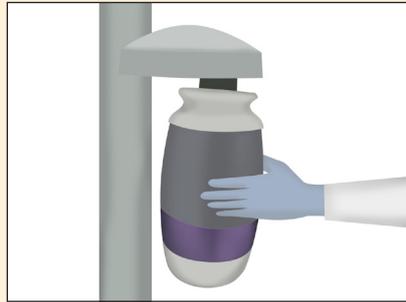

2. Connect bottle to the dental unit and flood the water pipes with GREEN&CLEAN BR.



3. Open all outlets and valves and wait until violet liquid exits the suppliers.



4. Close all outlets and valves and observe the action time (30 min).



5. Fill the bottle with clean water, reconnect to the unit and rinse the water lines completely.



6. Repeat rinsing process until the entire GREEN&CLEAN BR has been flushed out.

4% hydrogen peroxide solution for the cleaning, disinfection and biofilm removal in the water-bearing pipes of the dental unit with bottle systems

Characteristics

- effective biofilm removal in only 30 min
- simple application through colored solution
- recommended for the refurbishment of dental units as well as before starting a continuous water decontamination
- protection from hygienic-microbiological hazards for the dental staff and patients CE 0044

Recommendations for use

*IMPORTANT: Before performing the biofilm removal procedure, check whether the dentist element (e.g. instruments) is a stand-alone system or if it is connected to the assistant and patient elements (spitton bowl, saliva ejectors, etc.) of the dental chair: If it is a stand-alone system, use 250 ml of GREEN&CLEAN BR; if the dentist, assistant and patient elements are connected, use min. 1000 ml of GREEN&CLEAN BR

put on protective gloves and glasses before use!

remove all instruments (e.g. turbines, hand pieces, couplings) from the water consuming devices (except the air/water syringe) before use to prevent clogging for treatment units without a spitton bowl: use a suitable receptacle (such as a bucket or tub) to catch the escaping liquid

this process (shock treatment) should subsequently be repeated at least once a year and as necessary, even when continuously decontaminating the water pipe lines

Disinfecting the water bottle

to disinfect the water bottle, fill the bottle with GREEN&CLEAN BR up to the bottleneck

cover the bottle and swirl to ensure that all surfaces are wetted

observe the action time of 30 min

when the action time has elapsed, empty the bottle and rinse with clean water

if necessary, repeat this step several times until the entire GREEN&CLEAN BR has been flushed out

