

# GREEN&CLEAN CL

## Application



1. Fill the dosing cap up to one-fourth (= 5 ml of concentrate) with GREEN&CLEAN CL.



2. Fill container with cold water up to the indicator (= 600 ml of 0.8% dilution)



3. Hold small suction tube on opening no. 1 until air is sucked in (200 ml)



4. Hold large suction tube on opening no. 2 until air is sucked in (200 ml)



5. Empty the remaining solution into the cuspidor (200 ml)



6. Observe action time (60 min) and then rinse with cold water

## Cleaning and deodorization of suction systems (incl. spittoon bowl, amalgam separator and air-water separator)

### Characteristics

Concentrate for the daily cleaning of suction systems and amalgam separators especially recommended in case of increased contamination load or excessive foaming

very economical 0.8% concentration - 1 bottle lasts for 100 applications

high environmental compatibility

good material compatibility

aldehyde- and phenol-free

easy handling

CE

### Recommendations for use

1 x daily (evening) and after prolonged surgical treatments

Rinse the suction system with cold water after every treatment

Wear protective gloves when cleaning

Follow dosage directions exactly

Use mixed solution immediately

Observe the manufacturer's instructions for use

