

GREEN&CLEAN M2

Application



1. Press dispenser pump twice (= 6 ml of concentrate)



2. Fill container with cold water up to the indicator (= 600 ml of 1% dilution)



3. Hold small suction tube on opening no. 1 until air is sucked in (200 ml)



4. Hold large suction tube on opening no. 2 until air is sucked in (200 ml)



5. Empty the remaining solution into the cuspidor (200 ml)




6. Observe action time and then rinse (HBV/HIV/HCV: 60 min, tuberculocide: 240 min)

Dual, actively defoaming concentrate for the disinfection and deodorisation of suction systems and amalgam separators

Characteristics dual system (red/green) active against resistance bactericide, tuberculocide, fungicide (C. albicans), HBV/HIV/HCV highly concentrated and easy to dose excellent deodorising and protein dissolving power actively defoaming tested in accordance with EN standards and VAH guidelines

Recommendations for use 2 x daily (noon/evening) and after surgical treatments Rinse the suction system with cold water after every treatment Change colour after 83 applications (1 bottle) Wear protective gloves when disinfecting Follow dosage directions and recommended action times exactly Use mixed solution immediately Observe the manufacturer's instructions for use

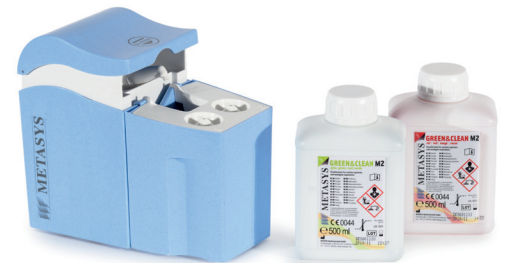

Refilling the dispenser



Remove the dosing unit from the dispenser and remove pump head by turning it counter-clockwise



Pour the new GREEN&CLEAN M2 into the container (500 ml)



Reattach pump head on the container by turning it clockwise, and put dosing unit back into the dispenser