
GREEN&CLEAN MK

METASYS

DE	Anleitung	NO	Instruksjoner
EN	Instructions	DA	Instruktioner
FR	Instructions	EL	Οδηγίες
IT	Istruzioni	PL	Instrukcje
ES	Instrucciones	CS	Instrukce
PT	Instruções	SK	Inštrukcie
NL	Instructies	SL	Navodila
SV	Instruktioner	HR	Instrukcije
FI	Ohjeet	HU	Utasítás
		ET	Juhendid
		LV	Instrukcijas
		LT	Instrukcijos
		RO	Instrucțiune
		IS	Leiðbeiningar

70

Desinfektionstücher
disinfecting wipes
teli desinfettanti
lingettes de désinfection

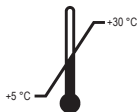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Extra-large disinfectant wipes

Teli desinfettanti extra grandi

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UFI: 1Y00-XOKS-E006-3QAX

Produktidentifikatoren / product identifiers:

Didecyldimethylammoniumchlorid [7173-51-5],
C12-14-Alkyl [(ethylphenyl)methyl] dimethylammoniumchlorid [85409-23-0],
Benzyl-C12-16-Alkyldimethylammoniumchlorid [68424-85-1]

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FR Extra-large disinfecting wipes. Alcohol-free disinfectant solution for disinfecting large surfaces of medical devices (e.g. alcohol-sensitive surfaces of dental units). Spectrum of efficacy/action times: HBV/HIV/HCV: 1 minute; bactericide, fungicide (C. albicans): 1 minute. Application: 1. Remove lid of dispenser bucket (security seal), place wipes inside. 2. Pour 750 ml from the canister or one bottle of the disinfectant solution GREEN&CLEAN SK evenly over the wipes. 3. Pull first wipe from the inside of the roll through the opening and close dispenser bucket. 4. Write the batch number and expiry date of the disinfectant solution on the enclosed label, fill in the rest of the information completely, and stick it onto the lid of the dispenser. 5. Ready to use as soon as all the liquid has been soaked up. This mixture is classified as not hazardous according to Regulation (EC) 1272/2008 [GHS]. Composition: <1% quaternary ammonium compounds

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METASYS Medizintechnik GmbH