
# GREEN&CLEAN MT

# METASYS

- DE** Achtung
- EN** Warning
- FR** Attention
- IT** Attenzione
- ES** Atención
- PT** Atenção
- NL** Waarschuwing
- SV** Varning
- FI** Varoitus

- NO** Advarsel
- DA** Advarsel
- EL** Προσοχή
- PL** Uwaga
- CS** Varování
- SK** Pozor
- SL** Pozor
- HR** Pozor
- HU** Figyelem
- ET** Hoiatu
- LV** Uzmanību
- LT** Atsargiai
- IS** Varúð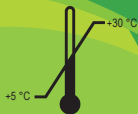

**Extragroße Desinfektionstücher**  
**Extra-large disinfectant wipes**  
**Teli disinfettanti extra grandi**  
**Suurikokoinen desinfectiopyyhe**

## 70

Desinfektionstücher  
 disinfecting wipes  
 teli disinfettanti  
 lingettes de désinfection

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**EN** Extra-large disinfecting wipes. Alcoholic disinfectant solution for disinfecting large surfaces of medical devices. Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, HSV-1): 30 seconds; bactericide, fungicide (C. albicans): 1 minute; tuberculocide: 5 minutes. Application: 1. Remove lid of dispenser bucket (security seal), place wipes inside. 2. Pour 750 ml from the canister or one bottle of the disinfectant solution GREEN&CLEAN SD evenly over the wipes. 3. Pull first wipe from the inside of the roll through the opening and close dispenser bucket. 4. Write the batch number and expiry date of the disinfectant solution on the enclosed label, fill in the rest of the information completely, and stick it onto the lid of the dispenser. 5. Ready to use as soon as all the liquid has been soaked up. **WARNING:** Flammable liquid and vapour. Causes serious eye irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Composition: 45% ethanol, 15% 2-propanol, 0.2% quaternary ammonium compounds, 0.2% glyoxal

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