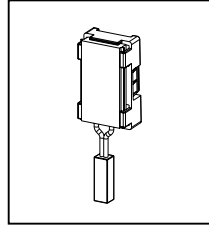
METASYS

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EN ▶ **Retrofit kit capacitive sensor
for COMPACT Dynamic**
Installation manual

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EN General information:

METASYS can only guarantee safety, reliability and performance of the apparatus, when the following instructions are adhered to:

- Installation or repairs are carried out by authorized service personnel in compliance with Standard IEC 601-1 (International Standard for medical electrical apparatus, especially part 1: General rules for safety).
- The apparatus is assembled, operated and maintained according to the instructions of COMPACT Dynamic provided. Only original parts are used for repairs or replacements. Every inspection and service intervention has to be recorded in the apparatus logbook.
- METASYS accepts no responsibility for damages caused due to external factors, such as wrong installation, use of false information, improper use of the apparatus or unauthorized technical intervention.

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EN 1 Delivery contents (Order number 50040042)

- Light barrier with capacitive sensor
- intermediate electronic circuit board
- plug

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EN Extraction from the separator (module 2) COMPACT Dynamic (see also Installation, Operation, Maintenance manual COMPACT Dynamic, page 10, point 1 to 2)

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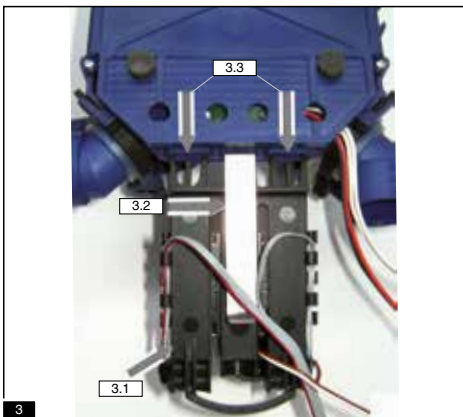
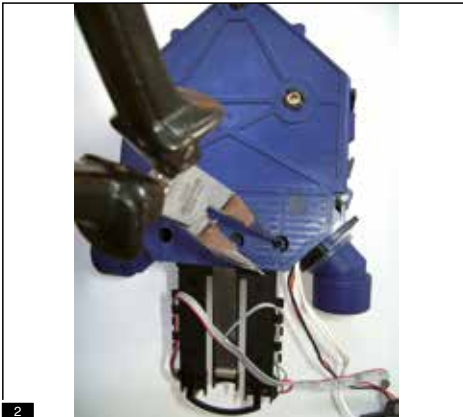
EN 3 Install the new light barrier with capacitive sensor **[3.1]** and metal clip **[3.2]** until the block **[3.3]** – check the fixation!

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EN 4 Place intermediate electronic circuit board at port „X5 LIS“ onto main circuit board

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EN 5 + 6 Connections

5.1 Connect cable of the intermediate electronic circuit board to main circuit board's connection cable at probe „X1 TS“ (white probe cable is no longer in use).

5.2 Connect cable of capacitive sensor on the intermediate electronic circuit board.

5.3 Connect cable of light barrier on the intermediate electronic circuit board.

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EN Check all other electronic connections on the circuit board!

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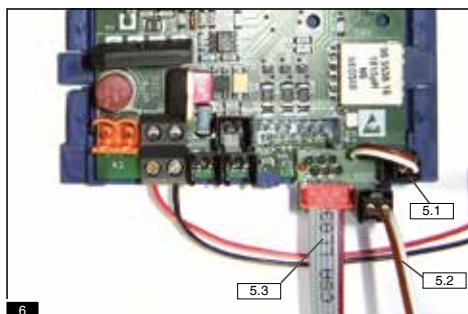
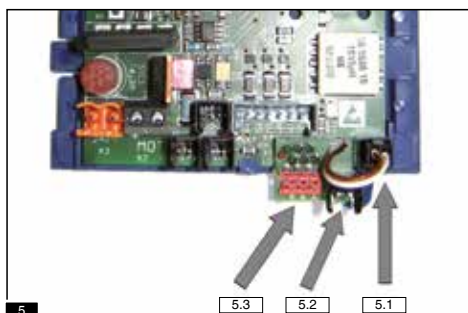
EN 7 Remove yellow probe on separator (including cable) and insert provided plug instead.

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EN Insert separator and switch device on! Carry out function test. (See Installation, Operation, Maintenance manual COMPACT Dynamic, page 10 - from the point 8)

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Medizintechnik GmbH